



## **Product: Universal Shoulder Immobilizer**

### **Model: 2800**

- Immobilizes arm and shoulder.
- Contoured chest strap fits both male and female.
- Unique chest strap immobilizes the upper arm eliminating the need for a humeral cuff.
- Shoulder strap and forearm cuff may be placed in desired position.
- Hook and loop closure for easy application.
- Fits either left or right arm.



#### **Order #**

2800

#### **Size**

Universal

#### **HOW TO APPLY:**

You will need assistance when initialing applying and removing. Wrap the waist band around your chest just under breast area and secure the closure – can be worn under clothing. Attach the hook and loop portion of the arm strap to the waist band under your injured arm. The strap should be around the mid portion of the upper arm. Wrap the wrist strap so that the hook will attach to the waist band. Bring your arm up to a comfortable position by bending your elbow. Attach the wrist strap to the front of the waist band. Place the shoulder strap over the shoulder of the arm that is not injured. Attach the hook end to the back of the waist band. Secure the slide buckle end to the front of the waist band and adjust, for your comfort. Adjust as needed.

#### **CARE INSTRUCTIONS:**

Hand wash in cold water with mild soap. Wrap in a towel to remove excess water (do not twist or wring). Air-dry away from heat. Do not tumble dry. Do not bleach.

#### **CAUTION:**

Wear as recommended. Always consult a physician when encountering persistent pain, discomfort, inflammation, or swelling.

**SUGG HCPCS CODE: L3651**