


# Never Ever use this product on or in your Ears -Never

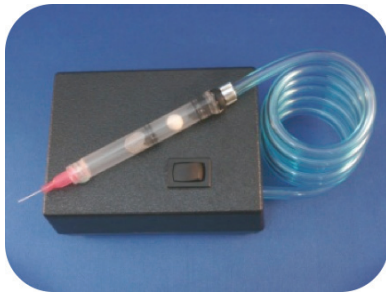

**1. Vacuuming Daily is Highly Recommended** , you may leave Wax traps in place, however, oils will go beyond wax trap and once a week you should remove wax trap and thoroughly vacuum the tube where the sounds comes out of.

Watch Video on line at [www.jodivac.com](http://www.jodivac.com) under "How it Works"


**2. We recommend consulting your Hearing Care Professional to show you where and how to vacuum your hearing aid.**

**3. Please consider the last time your Hearing Aid was serviced, I have been repairing Hearing Aids for some 15 years, it might be beneficial to follow step 2. But if you must 😊, the first vacuuming will require some extra care as wax may have built up, **go slowly**, paying attention to **keep needle clean!** A plugged needle will act as a ramrod pushing wax in deeper.**




**PROBLEM**-Pump is Running but the Ball stays Down- Your Needle is Plugged with Wax.


**SOLUTION** --

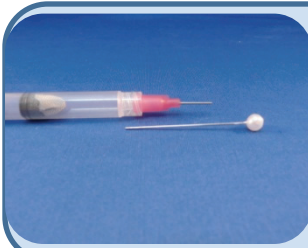

While Pump is Running insert Needle Reamer into Needle to Push Wax through Needle and into Filter Assembly.

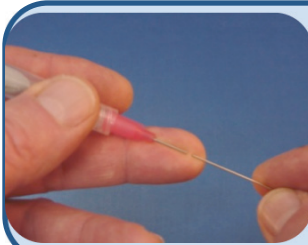

Lay Needle FLAT against your finger creating an Indent, slide reamer along Indent and into needle.

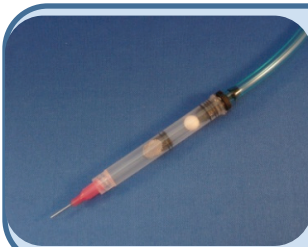

Remove Reamer and Ball will rise indicating air flow which is how it should be.

**4. Use no Force**, insert tip of needle into opening and slowly work your way down until you bottom out.

**5. When vacuuming is completed, you should store the reamer in the needle.** This ensures a clean needle to start with.

