

Arm Rest Pad with Pocket



Reorder #	Description	Size	Wt.	Unit
<input type="checkbox"/> 703018	Full arm wheelchair	16" long	.5 lbs	Set
<input type="checkbox"/> 703019	Half arm wheelchair	11" long	.4 lbs	Set

PURPOSE

Wheelchair armrests that are either padded or not are uncomfortable and expose the residents to pressure sores on the arm or elbow. As a result many residents avoid using armrests and place their arms on their lap which contributes to bad posture. This foam padded armrest attaches to any wheelchair and avoids arm pressure sores and skin abrasion by providing ample cushioning. In addition, one of the two armrests is provided with a mesh pocket or side pouch that provides a convenient place for personal items.

USE INSTRUCTIONS

Open up velcro strips and place the pads around the armrests with the pocket facing to the inside. Place the pocket on the side (right or left) that provides easier access to the resident. Utilize the 16" long set on a full arm wheelchair and the 11" set on a desk arm wheelchair.

CARE INSTRUCTIONS

The armrest cushions/handy bag may be washed and dried at temperatures not greater than 180°F (82°C). If bleach is used assure that it is diluted or mild.