



**Product: Sling and Swathe Immobilizer**  
**Model: 291X**

- Secures the arm against the body.
- Soft foam construction.
- May be used as an arm sling if swathe is not desired.
- Hook and loop closure for easy application.
- Fits either left or right arm.



| <b>Order #</b> | <b>Size</b> | <b>Swathe Length</b> |
|----------------|-------------|----------------------|
| 2913           | Small       | 34"                  |
| 2915           | Medium      | 40"                  |
| 2917           | Large       | 48"                  |
| 2918           | X-Large     | 54"                  |

**HOW TO APPLY:**

May need assistance when applying and removing. Place the injured arm in the pouch sling. Take the ends of both canvas straps over the top of the shoulders, around the back to the front and attach the hook portion of the strap to the front of the pouch to secure. Next, wrap the swathe around the injured arm, just above the bent elbow, and the chest. Secure the hook and loop closure on the swathe to hold the arm and shoulder in place. Adjust as needed.

**CARE INSTRUCTIONS:**

Hand wash in cold water with mild soap. Wrap in a towel to remove excess water (do not twist or wring). Air-dry away from heat. Do not tumble dry. Do not bleach.

**CAUTION:**

Wear as recommended. Always consult a physician when encountering persistent pain, discomfort, inflammation, or swelling.

**SUGG HCPCS CODE: L3670**