



**Product: Elastic Slip-On Knee Support**  
**Model: 32X Open Patella, 33X Closed Patella**

- Easy slip-on design.
- Designed to give compressive support to weak knees or after mild injury.
- Durable knitted elastic provides support with flexibility.
- Seamless construction will not irritate skin.
- Available in either open or closed patella design.
- 11" sleeve.



Order # Open Patella	Order # Closed Patella	Size	Knee Circumference
323	333	Small	12 ½" - 14 ½"
325	335	Medium	14" - 16"
327	337	Large	15 ½" - 18"
328	338	X-Large	17 ½" - 20"

**SIZING:** Measure around the center of the knee with leg extended.

**HOW TO APPLY:**

Pull the support up over the knee and center on the kneecap. The support should fit snug but not so tight to restrict circulation.

**CARE INSTRUCTIONS:**

Hand wash in cold water with mild soap. Wrap in a towel to remove excess water (do not twist or wring). Air dry away from heat. Do not tumble dry. Do not bleach.

**CAUTION:**

Wear as recommended. Always consult a physician when encountering persistent pain, discomfort, inflammation, or swelling.

**SUGG HCPCS CODE:** A9270