EN

With your inspiring rating, COSTWAY will be more consistent to offer you EASY SHOPPING EXPERIENCE, GOOD PRODUCTS and EFFICIENT SERVICE!

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Commode Chair / Chaise d'Aisance

Dear users:

- Thanks for buying our commode chair.
- Before use, please carefully read this instructions to ensure correct use.
- After reading, please keep the instructions for use properly and future reference at any time.

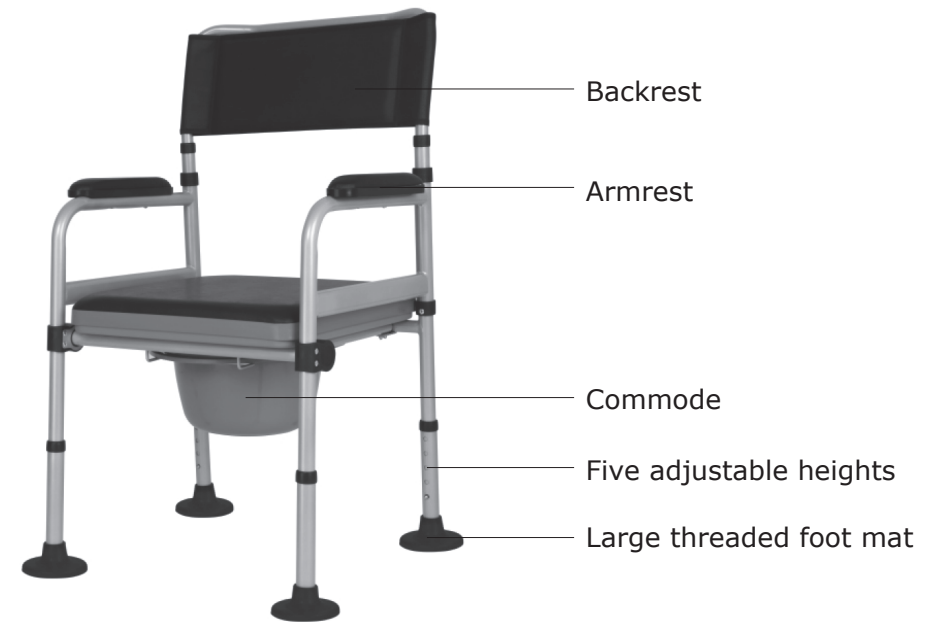
Product Introduction

- This product is an integrated multifunctional commode chair newly launched. By making return visits to regular customers, it is well designed by our designers for the elderly and people who have difficulty in moving. It can be put beside the bed and in the bathroom as a commode chair. The seat cushion and backrest are made of soft material and are comfortable to sit.
- The main structure of this commode chair is made of high-carbon steel with five adjustable height holes. The commode can be pulled out from the front and the back. More user-friendly design can solve the problems frequently encountered by the elderly in life.
- It uses thicker steel tube to satisfy the dependence and requirements of the elderly and pregnant women for commode chair. This foldable commode chair is really worth having.

Material: aluminium alloy+plastic

Sitting height: 17.7"~21.7" (from the seat cushion to the ground)

Net weight: 18lbs

Installation & Use Method

1. Open the package and take out the product.

2. Rotate the two armrests back to unfold the chair. Hold the chair frame with one hand, press down the ball with another hand, and then insert the horizontal tube into the mounting base.

3. Press down the ball on the backrest tube with two hands at the same time, insert the backrest into the frame from both sides at the same time, and ensure that the ball head is completely exposed outside the hole.

4. Buckle the seat onto the horizontal tube of frame.

5. Put in the commode which can be pulled out from the front and the back.

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