



Technical Information Clorox Healthcare® Bleach Germicidal Wipes



OVERVIEW

Clorox Healthcare® Bleach Germicidal Wipes are engineered as a ready-to-use sporicidal cleaner-disinfectant system for healthcare facilities.

These EPA-registered disinfectant wipes contain sodium hypochlorite and other ingredients to kill *C. difficile* spores and other healthcare-relevant pathogens in 3 minutes or less on a variety of hard, nonporous healthcare surfaces.




INGREDIENT INFORMATION

Active ingredient: sodium hypochlorite.....0.55%*
 Other ingredients:.....99.45%
 Total (not including weight of wipe):.....100%
 *0.52% (5200ppm) available chlorine

At a Glance Clorox Healthcare® Bleach Germicidal Wipes	
EPA Registration Number	67619-12
Product Classification	Pre-moistened wipes
Dilution	Ready to use
Disinfectant Type	1-step cleaner-disinfectant
Active Ingredient Chemistry	Sodium Hypochlorite (Bleach) (1:10 dilution of bleach) 0.55% (5500 ppm)
Odor	fruity, floral, bleach
Shelf Life	Expiration date listed on each package: month, day, year
Toxicity Signal Word	Caution
HMIS Rating	1-0-1-A
PPE Required	Follow your facility's protocols

AVAILABLE FORMATS

Clorox Healthcare® Bleach Germicidal Wipes are available in multiple sizes and formats to accomplish different disinfecting jobs.

	Designed for	Item #	Quantity	Shelf Life
Clinical Wipes (5"x6") 	Daily Disinfection Small high-touch surfaces Medical equipment	30577	150 ct. canister, 6/case	12 months
Multi-Purpose Wipes (6.75"x9") 	Daily Disinfection	35309	70 ct. canister, 6/case	14 months
	Patient room surfaces	31469	20 ct. soft pack, 24/case	12 months
	Medical equipment	31424	50 ct. carton, 6/case	12 months
Terminal Wipes (12"x12") 	Discharge Disinfection	30358	110 ct. bucket, 2/case	12 months
	Large areas	30359	110 ct. refill, 2/case	12 months

Use products as directed.

© 2017 Clorox Professional Products Company

NI-38488

Updated June 2017



Technical Information

Clorox Healthcare®

Bleach Germicidal Wipes



DISINFECTION DIRECTIONS FOR USE#



1. Don PPE, then REMOVE gross soil if visible.* When disinfecting surfaces contaminated with *C. difficile* spores, always clean surfaces prior to disinfecting.



2. WIPE surface with wipe until completely wet.



3. To DISINFECT, allow surface to remain wet for contact time listed below:
Bacteria - 30 seconds
Viruses - 1 minute
C. difficile spores - 3 minutes



4. WIPE surface with a clean, damp cloth to prevent residue buildup.

Special Instructions for Cleaning Prior to Disinfection Against *Clostridium difficile* spores:

- **Personal Protection:** Wear appropriate barrier protection such as gloves, gowns, masks and eye covering.
- **Cleaning Procedure:** Fecal matter/waste must be thoroughly cleaned from surfaces/objects before disinfection by application with this product. Cleaning is to include vigorous wiping and/or scrubbing, until all visible soil is removed. Special attention is needed for high-touch surfaces. Surfaces in patient rooms are to be cleaned in an appropriate manner, such as from right to left or left to right on horizontal surfaces, and top to bottom on vertical surfaces to minimize spreading of the spores. Restrooms are to be cleaned last. Do not reuse soiled cloths.
- **Infectious Materials Disposal:** Materials used in the cleaning process that may contain feces/waste are to be disposed of immediately in accordance with local regulations for infectious materials disposal.

Special Instructions for Using This Product to Clean and Decontaminate Against HIV-1, HBV and HCV on Surfaces/Objects Soiled with Blood/Body Fluids:

- **Personal Protection:** When handling items soiled with blood or body fluids, use disposable impervious gloves, gowns, masks and eye coverings.
- **Cleaning Procedure:** Blood and other body fluids must be thoroughly cleaned from surfaces and other objects before applying this product.
- **Contact Time:** Allow surfaces to remain wet for 1 minute, let air dry. For all other organisms, see directions for contact times.
- **Disposal of Infectious Materials:** Use disposable impervious gloves, gowns, masks and eye coverings. Blood and other body fluids must be autoclaved and disposed of according to local regulations for infectious waste disposal.

This product is not to be used as a terminal sterilant/high-level disinfectant on any surface or instrument that **(1)** is introduced directly into the human body, either into or in contact with the bloodstream or normally sterile areas of the body, or **(2)** contacts intact mucous membranes, but which does not ordinarily penetrate the blood barrier or otherwise enter normally sterile areas of the body. This product may be used to preclean or decontaminate critical or semi critical medical devices prior to sterilization or high-level disinfection.

See product label for complete instructions. This is a one-step product. Precleaning is required only if blood, vomit, fecal matter or other soil is visible, or in advance of disinfecting *C. difficile* spores.