



Low Vision Chart Stands Assembly Guide


Part Number: 607500, 607600



1. Screw stand into base plate



2. Unscrew top cap of the stand

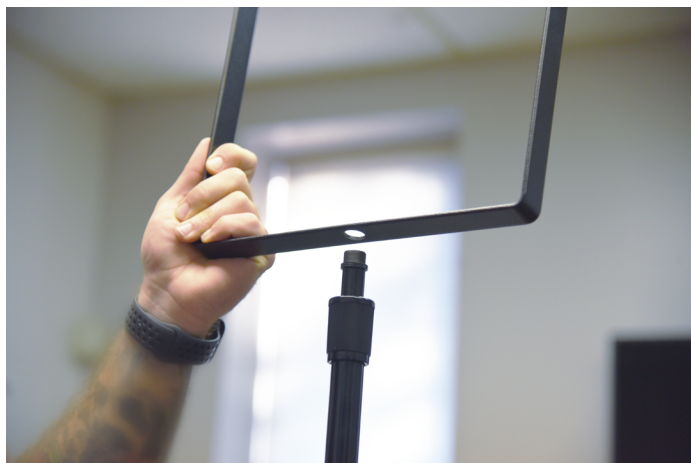


3. Insert top of stand into the bottom hole of the Low Vision Book bracket



Low Vision Chart Stands Assembly Guide

Part Number: 607500, 607600

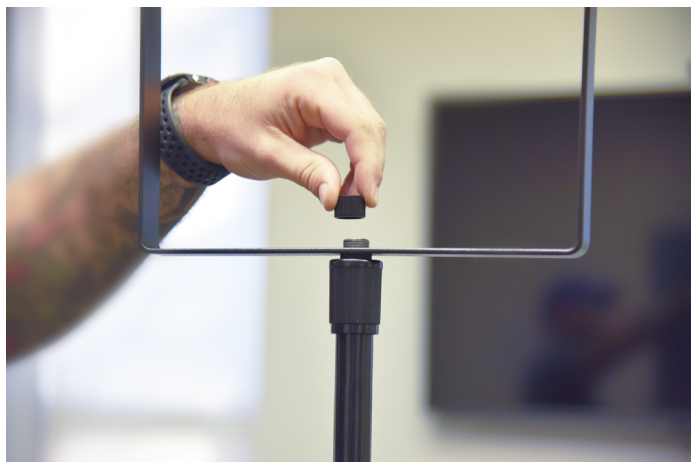

4. Screw top cap back on stand tightly to secure the Low Vision Book bracket



5. Remove one black cap off of the stainless steel rod



6. Line up the stainless steel rod to the hole on the top of the mounting bracket. Push through the top spiral bound of the Low Vision book and hole on the other side of the mounting bracket



Low Vision Chart Stands Assembly Guide

Part Number: 607500, 607600



7. Re-insert black cap onto the stainless steel rod



8. Unscrew knuckle on the stand to adjust to the desired height. Tighten knuckle to secure in place.



9. This guide also works for the table stand version (#607600).