

Computer Anaglyph Screen Filter

Part Number: 713100

Guide:

Use only as shown by your Eye Professional.

Computer Attachment:

Use of the screen in a person's lap is not recommended. Tilt the computer screen and then prop the filter against it. If the filter will not stay resting against the computer or tablet screen, you might use photographer's putty on the bottom to hold in place.

Cautions:

Treat the filter screen with respect. It is not glass but it can shatter if dropped. Avoid the corner ends of the black U-channels as these are somewhat sharp. Do not remove the channels.

Cleaning:

Do not use alcohol or ammonia. Acrylic can easily become scratched on the surface from harsh cleaners. This can produce small scratches on the surface of the glass, resulting in a cloudy look. If there is just a small amount of dust, use a small amount of water on a very soft cloth.

If there is more significant dirt or fingerprints, use a mix of a mild, non-abrasive (dish washing) detergent with the water. Use this mix with a non-abrasive/non-contaminating cloth. You can also use Novus #1™ or Brilliance™.

Do not let any excess liquid seep in to the edges or the "seam". Do not spray. Turn the cloth often. Avoid rubbing. Use a second cloth to dry.

