

Model SA-40

It's as easy as 1,2,3

Quick Start Guide

HearAll[™]

Cell Phone Amplifier





Pairing the SA-40 with Your Cell Phone:

1. To perform the initial pairing, activate the *Bluetooth*^{®*} feature of your cell phone.
2. Turn on the SA-40. It will blink as it pairs to your cell phone. The SA-40 will remain paired until you delete it from your cell phone.



Select an Operating Mode:

1. **HS** – Use as a handset.
2. **T** – Couple with your T-Coil hearing aid.
3. **SPKR** – Use as a speakerphone.

Answer a Call on the SA-40:

1. When your phone rings, press  to answer.
2. Adjust volume by pressing the **VOL (+)** or **VOL (-)** buttons.
3. To enhance clarity, adjust tone by moving **TONE**, until you hear the best.
4. Press  to hang up.

Making a Call on the SA-40:

1. Press  to start.
2. If your cell phone is equipped with voice-dial, speak the desired name or number into the SA-40.
3. If your cell phone does not have voice-dial, enter the number using key pad of your cell phone.
4. Press  to hang up.

Please Note:

The SA-40 amplifies your caller's voice up to 100 times louder than your cell phone. Use care when setting phone volume. The SA-40 should always be set to (HS) Handset Mode when handset is pressed to the ear.

Need Help? Call Toll-free Help Line:
866-376-9271 (9AM - 5PM PST, M-F)

SERENE[®]

Serene Innovations, Inc.
14731 Carmenita Road, Norwalk, CA 90650
www.sereneinnovations.com

*Bluetooth[®] is a trademark of Bluetooth SIG, Inc.