

**Installation**

1. Mount the magnet on the door or window so that it clears the door jamb or window frame and does not interfere with normal door or window operation.
2. Mount the magnetic switch so that it is as close to the magnet as possible when the door or window is closed. Spacing should be less than approximately ¼”.
3. Mount the transmitter to the wall or frame with the supplied Velcro strips. You can also hang the transmitter on a nail or screw using the opening in the back of the case. Do not mount the transmitter itself to a metallic surface or range will be severely reduced.



**Operation:**

1. When the door or window moves enough that the magnet is separated from the switch, the red LED on the transmitter will light up. The transmitter will then send a signal to any Silent Call Signature Series Receiver activating the receiver. The transmitter will transmit for approximately five seconds and then stop until the door or window is closed. The door or window must be closed for approximately five seconds before the transmitter will reset and be ready to activate again.
2. Transmission range is determined by which Signature Series receiver you are using.
3. This unit is powered by two AA alkaline batteries (included) which should last a year or more, depending on use.
4. There is a yellow LED (low battery indicator) on the face of the transmitter that will light up when the battery is low and needs to be changed.

**Transmitter Settings:**

The Silent Call Medallion system is digitally encoded. All Silent Call Medallion Series receivers are capable of receiving one million different codes and the transmitters are preset at the factory to send one of the million different codes. Please refer to your receiver manual for instruction on teaching the transmitter code to the receiver.

**Replacement Batteries:**

Use two AA 1.5V Alkaline Batteries. AA Batteries may be purchased from local retail stores. Transmitter is shipped with 2 Rayovac Alkaline Batteries Type LR6. Batteries may be purchased Online at [www.silentcall.com](http://www.silentcall.com).

**Technical Support**

For technical support on this or any other Silent Call product, please feel free to contact us. You can reach us by phone at 800-572-5227 (voice or TTY) or by Email at [support@silentcall.com](mailto:support@silentcall.com)

**Limited Warranty**

Your transmitter is warranted to be free of defects in material and workmanship for five years from the date of initial purchase. During that time, the unit will be repaired or replaced free of charge when shipped prepaid to Silent Call Communications. This warranty is void if the defect is caused by customer abuse or neglect.