

Model MAT4-MC

TransMATTer Transmitter 418MHz

Installation:

1. Place the mat where it will be stepped on anywhere except the very edge. The water resistant material used to manufacture the TransMATTer is required to have a protective cover over it; this would usually consist of a door rug or similar covering.
2. The transmitter that is connected to the TransMATTer must not be mounted directly to a metallic surface. Doing so will severely reduce the range of the transmitter.
3. The TransMATTer is intended for indoor use only.

Operation:

1. Whenever the mat is stepped on, the Red LED on the face of the TransMATTer Signature Series transmitter will light up. The transmitter then sends a signal to any Signature Series Receiver, activating the Receiver and illuminating the Doorbell LED and Indicator symbol.
2. Transmission range is determined by which Signature Series receiver you are using, and will vary from 250 feet up to 2000 feet.
3. If pressure is maintained, the transmitter will continue to transmit until pressure is removed from the mat.

CAUTION: *When handling or shipping the TransMATTer, do not bend the mat sharply or fold it. It must be kept flat or rolled loosely or it will be damaged.*



Transmitter Settings

The Silent Call Medallion system is digitally encoded. All Silent Call Medallion Series receivers are capable of receiving one million different codes and the transmitters are preset at the factory to send one of the million different codes. Please refer to your receiver manual for instruction on teaching the transmitter code to the receiver.

Installation and Operating Instructions

Model MAT4-MC

TransMATTer Transmitter 418MHz

Replacement Batteries:

Use two AA 1.5V Alkaline Batteries. AA Batteries may be purchased from local retail stores. Transmitter is shipped with 2 Rayovac Alkaline Batteries Type LR6. Batteries may be purchased Online at www.silentcall.com.

Technical Support:

For technical support on this or any other Silent Call product, please feel free to contact us. You can reach us by phone at 800-572-5227 (voice or TTY) or by Email at support@silentcall.com

Limited Warranty:

The transmitter itself is warranted to be free of defects in material and workmanship for five years from the date of initial purchase. The pressure sensitive mat is warranted for one year. During that time, the unit will be repaired or replaced free of charge when shipped prepaid to Silent Call Communications. This warranty is void if the defect is caused by customer abuse or neglect.



Return for Serving To:

Silent Call Communications

5095 Williams Lake Road, Waterford Michigan 48329

(800) 572-5227 v/tty (248) 673-7360 fax

Website: www.silentcall.com Email: silentcall@silentcall.com