# SuperEar®

Model SE5000

## Personal Sound Amplifier

### Owner's Manual

El manual del propietario  
Le manuel du propriétaire  
Bedienungsanleitung  
Manuale del proprietario

#### Instructions for Use

1. To remove the battery compartment cover, gently depress the cover with your thumb and slide it in the direction of the arrow.
2. Insert the AAA 1.5v battery. Match the positive (+) and negative (-) ends to those indicated on the battery cavity. Alkaline batteries will deliver 2-3 times the operational life of lesser batteries.
3. Slide the battery compartment cover back into locked position.
4. Insert the earphone connection into the jack.
5. Be sure the amplifier switch is in the OFF position before putting on the earphones. Adjust headband to fit comfortably.
6. Slowly turn the volume control wheel to the desired level of amplification.
7. Attach the pocket clip to the bottom of the amplifier using the screw provided. Do not over-tighten. Can be used on pocket, belt, or other gear.

#### User Information

1. This amplifier has a maximum gain of 50dB. At approximately ¾ open on the volume control, the device is at full volume. If you turn the volume control all the way up, it may result in an annoying feedback noise. Simply turn the volume control down to eliminate feedback.
2. If headphones are placed near the amplifier while it is turned on, feedback squeal will result. Always turn your SuperEar® off before you remove your headphones.
3. The SuperEar® requires only a single 1.5AAA alkaline battery for up to 30 hours of operation. Battery included.
4. Premium stereo headphones and earbuds deliver clear, crisp sound particularly well in the human voice range. The foam ear pads are easily replaceable.

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## 3 YEAR WARRANTY

Sonic Technology Products, Inc. warrants to the original purchaser that your SuperEar® will be free of defects in material or workmanship for a period of 3 years from the date of purchase. Warranty is void if defects are caused by negligence, abuse, misuse or damage caused by water or heat.

This warranty gives you specific legal rights and you may have other rights which may vary from state to state.

Please call us first to determine if it will be necessary to send your SuperEar® in for service under the terms of this warranty. If yes, it must be returned in a well-packaged carton and shipped postage prepaid.

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