Scan to see how  
LightOn works!

# LightOn

## USER MANUAL

DreamZon

DreamZon Ltd.  
[www.dreamzon.com](http://www.dreamzon.com)  
[www.facebook.com/dreamzon](https://www.facebook.com/dreamzon)  
[www.youtube.com/DreamZonTubes](https://www.youtube.com/DreamZonTubes)  
E-mail: [info@dreamzon.com](mailto:info@dreamzon.com)

Copyright© 2012 DreamZon Ltd. All rights reserved.

Other product and company names mentioned herein may be trademarks or trade names of their respective owners.

To the maximum extent permitted by applicable law, under no circumstances shall DreamZon or any of its licensors be responsible for personal injury or any incidental, special, indirect or consequential damages, howsoever caused.

The third-party applications products and solutions provided with your device may have been created and may be owned by persons or entities not affiliated with or related to DreamZon. DreamZon does not own the copyrights or intellectual property rights to these third-party applications, products and solutions. As such, DreamZon does not take any responsibility for end-user support, functionality of the applications, or the information in the applications or these materials. DreamZon does not provide any warranty for the third-party applications, products and solutions.

By using the applications you acknowledge that the applications are provided as is without warranty of any kind, express or implied, to the maximum extent permitted by applicable law.

**For any inquiries or information, please contact DreamZon at [info@dreamzon.com](mailto:info@dreamzon.com), or visit us at [www.dreamzon.com](http://www.dreamzon.com)**

## **Table of Contents**

Safety Precautions and Important Notices	4
About LightOn	5
Unpacking	6
Getting Started	7
- Battery Installation	7
- Preparing the Mobile Phone for Use	7
LightOn Unit Layout	8
Using the LightOn Cradle	10
- Light Panels	10
- Resetting the Light Indicator	10
- Battery Level Indication (Self test)	10
- Low Battery Indicator	11
Operating Environment	11
External Alerting and Triggering Systems	12
Frequently Asked Questions	14
Troubleshooting	16

## **Safety Precautions and Important Notices**

- Do not try to connect the LightOn cradle to electrical source! LightOn is operated by batteries and should not be connected to external AC power or wall socket in any case.
- Do not use cradle on surface that vibrates (e.g. dryer, washing machine, etc...). LightOn should only be placed on flat non-vibrating surfaces, for example: table, shelf, countertop etc.
- Do not use near water – cradle is not waterproof.
- Keep the LightOn unit away from any heat source as it may damage the unit.

This device complies with part 15 of the FCC Rules. Operation is subject to the following two conditions: (1) This device may not cause harmful interference, and (2) this device must accept any interference received, including interference that may cause undesired operation.

## About LightOn

LightOn is a unique decorative solution providing visual light notification for incoming SMS, Voice or Video calls, thus alerting its users that someone is trying to reach them. When a mobile phone is placed in the LightOn cradle, all incoming SMS messages or phone calls trigger a brightly flashing white LED light. This light continues to flash until the device is reset, notifying users on the missed message or call. If not manually reset after 3 hours, LightOn will stop blinking automatically, to preserve battery life.

The LightOn cradle's back-rest is designed to facilitate the use of sign language via mobile phones. The angle of the cradle enables you to place your phone in the cradle and then comfortably sign with both hands (e.g. using ASL or any other sign language).

LightOn allows simple connectivity to popular external signaling systems (purchased separately), including Sonic Alert DB100, Humantechnik Lisa, Bellman® Visit, Phonic Ear® Puzzle, Serene Innovations CentralAlert™, HGT VisiBel, Tokyo Shinyu Silwatch and GHE-CES AVISO Bee.

**Important Note:** Please check with your local reseller how to connect to your wireless alerting system (if any), and which specific 3<sup>rd</sup> party equipment is supported.

## Unpacking

Your package contains the following items:

LightOn Cradle

User Manual

Cable

Connectors (x2)

Batteries (x3)

## Getting Started

### Battery Installation


1. Press firmly downwards on the battery case cover, on the sign 
2. Insert three AA batteries, according to the (+) sign.
3. Click the battery case cover closed.

### Preparing the Mobile Phone for Use

To use the LightOn cradle:

1. Place the mobile phone on vibrate mode. If the phone has multiple vibration modes, select the mode that uses the strongest and longest vibration time.
2. Gently place phone in cradle.  
**Note:** If the phone is not placed gently, the movement may register as a vibration, causing LightOn to light up. If this happens, press the *Reset* button.

## LightOn Unit Layout

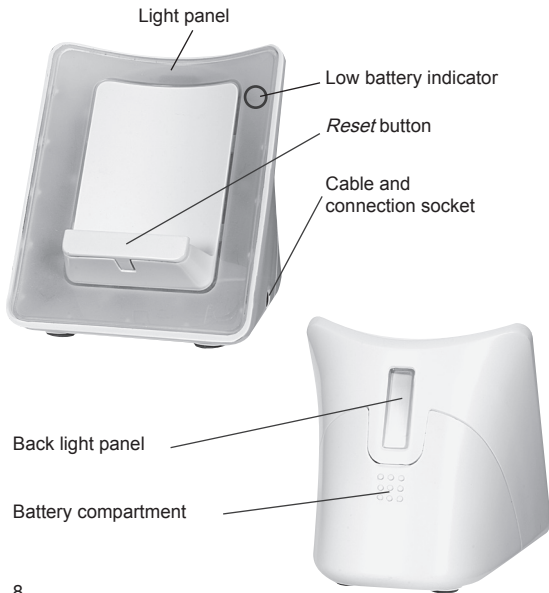Element	Function
Battery compartment	Holds the three AA batteries.
Low battery indicator	Red light will blink, indicating that batteries need to be replaced.
Reset button	Resets the light panel after a call or SMS has come in.
Light panel	Flashes repeatedly when the phone receives a call or SMS.
Back light panel	Flashes repeatedly when the phone receives a call or SMS.
Cable and connection socket	Connects LightOn to external alerting systems.

## Using the LightOn Cradle

### **Light Panels**

When the mobile phone on the LightOn cradle receives an SMS or phone call, white LED lights begin flashing brightly in the front and back panels.

For the first two minutes after a call or SMS comes in, the light flashes rapidly. After two minutes, the pace of the flashing decreases, conserving battery power, until it totally disappears after 3 hours.

### **Resetting the Light Indicator**

After receiving a call or SMS, you should reset the light indicator, so that you know if a new call or SMS comes in.

To reset the light indicator, simply click the *Reset* button in the front of the device. The device is now reset.

### **Battery Level Indication (Self Test)**

Use the “Self Test” function to verify that LightOn is powered on, and the batteries are working properly: While LightOn is idle, simply press the *Reset* button. LightOn will blink twice to indicate that the unit is working well.

## **Low Battery Indicator**

An incessant flashing in red indicates that battery power is weak. When battery power becomes too weak for the device to operate, it switches off automatically. Please replace the batteries if the Battery Level indicator flashes fast repeatedly, otherwise the LightOn device may not function well.

## **Operating Environment**

The LightOn phone cradle should be placed on a level, non-vibrating surface.






Since LightOn's technology is based on vibration sensors, the cradle's operation is designed to have minimum disturbances of radio frequency signals or other mobile phones in the vicinity.

## **External Alerting and Triggering Systems**

LightOn can connect to external triggering and wireless signaling systems, thus enabling better communications and improving day to day quality of life.

LightOn supports today connectivity to market leading wireless signaling systems (purchased separately), including Sonic Alert DB100, Humantechnik Lisa, Bellman® Visit, Phonic Ear® Puzzle, Serene Innovations CentralAlert™, HGT VisiBel, Tokyo Shinyu Silwatch and GHE-CES AVISO Bee.

To connect to an external alerting system, connect one end of the provided cable into the LightOn cradle's connection socket, and the other end into the external system device connection socket (use one of the supplied connectors, if needed). For example, connect LightOn to a transmitter, or a vibrating pillow (bed shaker).

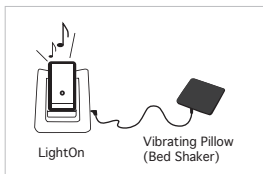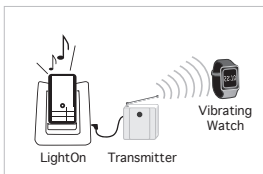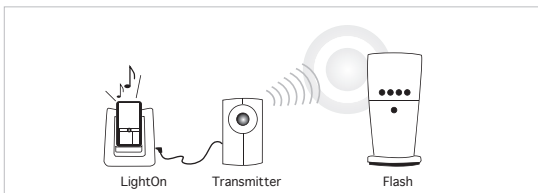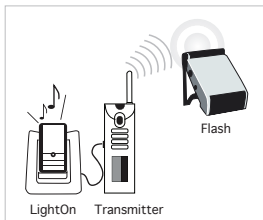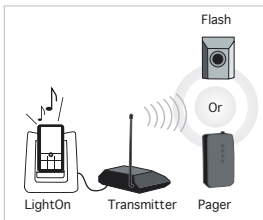## Frequently Asked Questions

**Q: Is my mobile phone supported by LightOn?**

**A:** LightOn supports any mobile phone which has vibration mode, which basically means all the phones in the market. For example, LightOn supports iPhone, Blackberry, HTC smartphones, Samsung and LG smartphones, Android devices, Nokia and Motorola phones, and many more.

**Q: Where can I purchase LightOn?**

**A:** For the latest distributor list, by country, please refer to the DreamZon web site: [www.dreamzon.com](http://www.dreamzon.com)  
LightOn can also be purchased online on the DreamZon web site.

**Q: How long will LightOn batteries last?**

**A:** LightOn was designed for efficient battery use and full portability. Based on average usage, LightOn's batteries should last more than 6 months.

**Q: Can I connect LightOn to an external power source?**

**A:** NO! Do not connect LightOn to any kind of power source. LightOn is designed for full portability, therefore it's operated by 3 batteries only. You can easily carry LightOn along with you when you travel!

**Q: Does LightOn charge the batteries?**

**A:** No, since LightOn does not connect to power, it does not charge the 3 batteries.

**Q: Can LightOn charge my mobile phone?**

**A:** No, LightOn does not charge the mobile phone, however while your mobile phone is placed on the LightOn cradle, it can be charged – just connect the phone's charging cord directly to external power. Please see the picture album explaining this functionality on our Facebook page: [www.facebook.com/dreamzon](http://www.facebook.com/dreamzon)

**Q: Where can I purchase the external 3rd party wireless alerting systems?**

**A:** For the latest information about external systems supported by LightOn, please contact your local reseller: [www.dreamzon.com](http://www.dreamzon.com) or, contact DreamZon at [info@dreamzon.com](mailto:info@dreamzon.com)

**Q: What is the price of LightOn?**

**A:** For the latest pricing information, please contact your local reseller: [www.dreamzon.com](http://www.dreamzon.com) or, contact DreamZon at [info@dreamzon.com](mailto:info@dreamzon.com)

## **Troubleshooting**

### **Light is not activated by SMS or phone call:**

- Check that the batteries are inserted properly and that they do not need to be changed.
- Check that the phone is on the longest and strongest vibrating mode.
- For Android phones - try downloading “Vibrate Tools” app from Google Play, to change your vibration settings to stronger and longer vibration.
- The mobile phone needs to be centered on the cradle, as close to the back of the LightOn cradle as possible, in order to achieve maximum contact between the phone and the cradle sensor.

### **Light flashes even though no incoming call:**

- Cradle placed on uneven or vibrating surface (e.g. next to keyboard).
- Cradle is placed very close to strong RF system.
- Push the *Reset* button and place the Mobile phone again.
- Mobile phone was not placed gently in cradle – simply press the *Reset* button.

### **Red LED is constantly flashing:**

- Replace batteries.

### **LightOn doesn't trigger the external wireless alerting system:**

- Make sure that the cable and/or connector are properly inserted on both sides.  
If the problem remains, please contact your reseller.

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Copyright© 2012 DreamZon Ltd. Alle Rechte vorbehalten.
Copyright© 2012 DreamZon Ltd. Tous droits réservés.
Copyright© 2012 DreamZon Ltd. Todos los derechos reservados.