
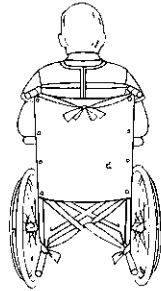


# SLEEVED JACKET

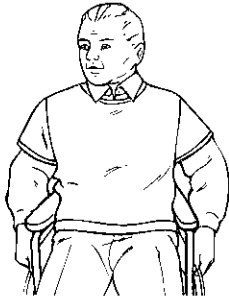

Reorder #	Fabric	Size	Unit
<input type="checkbox"/> 304203	Polyester	Small	1
<input type="checkbox"/> 304204	Polyester	Medium	1
<input type="checkbox"/> 304205	Polyester	Large	1
<input type="checkbox"/> 304213	CooLine Mesh	Small	1
<input type="checkbox"/> 304214	CooLine Mesh	Medium	1
<input type="checkbox"/> 304215	CooLine Mesh	Large	1

**Caution:** Federal law restricts this device to sale by or on the order of a physician or licensed health care professional.

**Indications** - The Skil-Care Sleeved Jacket is recommended for use with a patient who:

- Requires posture control while seated in the wheelchair
- Has a history of easily removing vest-style and poncho-style restraints

**Contraindications for Use** - Do not use this device:

- For a patient who slides down in the wheelchair
- For a patient in a bed, geri-chair, or lounge chair. This device is intended for wheelchair use only

## Application

1. Select proper size for your patient.

### Sizing Table for Skil-Care Products (see note)

Binding Color	Size	Weight (lb./kg)
Red	Small	80-130/26-59
Green	Medium	130-160/59-73
Yellow	Large	160-200/73-91

**Note:** Device size to patient weight relationships are approximate. Another criterion for device size selection may include patient torso size. Improper size selection may be indicated by difficulty experienced when closing zipper or shoulder seams that fall two or more inches below patient's shoulder.

2. Before seating patient in wheelchair, make certain that both brakes are locked and both footrests are positioned to the sides of the chair. Seat patient as far back in the wheelchair as possible. Position footrests to the front of the chair and place patient's feet on them.
3. Put jacket on patient making certain that the zipper is behind the patient. Never apply a sleeved jacket with the zipper in front of the patient.
4. Close the zipper.
5. Behind the wheelchair, pass the waist straps through the space between the wheelchair backrest and seat.
6. Cross the straps, wrapping one around the other as shown in illustration A.
7. Position left strap at right kick-spur and right strap at left kick-spur as shown in illustration B.
8. Wrap the strap several times around the kick-spur as shown in illustration C. Leave some excess so that you can proceed to next step.



Illustration A

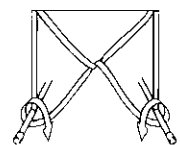


Illustration B

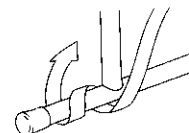


Illustration C

See **important warnings and cautions** on reverse side.

9. Wrap the strap around the vertical bar over the kick spur. Note that the strap is also wrapped around itself. See illustration D.

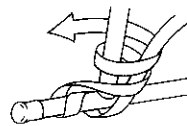


Illustration D

10. Tuck the loose end under the last wrap and pull down to secure. See illustration E. This creates a knot-free closure.

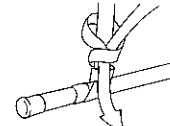


Illustration E

11. With patient sitting erect, wrap the left shoulder strap around the left push handle and the right shoulder strap around the right push handle. Leave enough excess so that the ends may be tied together in a bow. See illustration F.

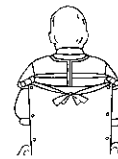


Illustration F

## Laundering


This device should not be laundered or dried at temperatures that exceed 180° F (82° C).

Do not use bleach.

See **important warnings and cautions** below.

### Always...


Always inspect condition of device.

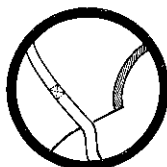

Always read package insert to determine if device is intended for wheelchair and/or bed use.



Always monitor patients in restrictive devices. Make certain to follow your facility's procedures for periodic release, toileting, and exercise.

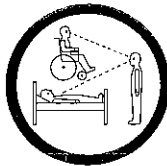

### Never...

Never use a device that has worn or frayed fabric or webbing. Discard damaged units immediately. Do not alter products.



Never use a Skil-Care device as a restraint in a motor vehicle.



Never place a patient wearing a restrictive device in a place where he/she is not easily monitored.



(Date of issue: 9/3/96)

**Skil-Care**<sup>TM</sup>  
CORPORATION

*Advanced design for nursing and therapy.*