



**NEW IMPROVED
Bilateral Anti-Rotation
Gripper**

Patent Pending

Original
Skil-Care™
CORPORATION
Design



Heel-Float//
sold separately
ReOrder # 503134

Re-Order Number	Description	Size	Units
503098	Bilateral Anti-Rotation Gripper™	11 3/8" x 8" x 3 1/2"	Each
503038	Plastic Bilateral Anti-Rotation	Large/Bariatric	Each
503039	Plastic Bilateral Anti-Rotation	Small/Medium	Each

Purpose

In the event it is advantageous to maintain the leg in a neutral position when lying in bed this bilateral anti-rotation gripper is recommended to be used in conjunction with most Skil-Care® heel boots.

Easy to Use

First place the appropriate Skil-Care® heel boot on the patient, then place the foot and boot into the Bilateral Anti-Rotation Gripper while lying on the bed. The Bilateral Anti-Rotation Gripper will release from the boot when the boot is lifted out of the Bilateral Anti-Rotation Gripper.

Easy Care

The Bilateral Anti-Rotation Gripper can be wiped clean with soap and water. If disinfected, avoid using chlorine (bleach) chemicals. Do Not Launder.

This product is guaranteed for 6 months.