
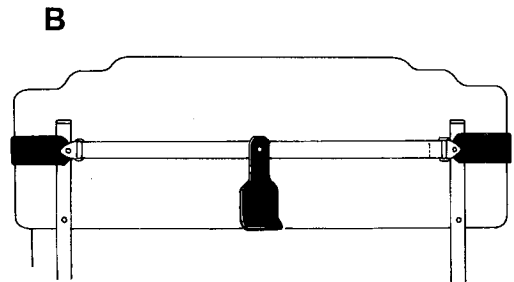


## Bed Alarm Mounting Bracket and Belt

Reorder #	Size	Unit
□909399	Fits all standard hospital-style beds	Each

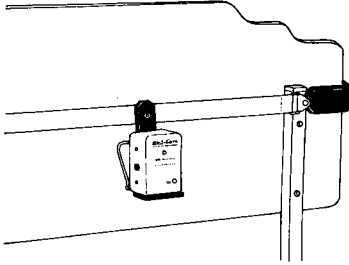

### Purpose

Skil-Care's Bed Alarm Bracket and Belt is designed to discourage patient tampering with bed alarm units. It permits bed alarm units to be mounted on the back of headboards, away from the patient's easy reach. This unit may also be used for personal alarms used in both beds and wheelchairs.

### Application (For all personal alarms and bed alarms other than those made by Skil-Care)

1. Place one hook over a side edge of the headboard. (Illustration A)
2. Stretch the elastic belt and place the other hook over the opposite edge of headboard. (Illustration B)
3. Remove the backing from the enclosed Velcro<sup>®</sup> strip and attach the strip to the back of the alarm unit. Place the alarm unit to the loop material on the bracket.

### Application for Skil-Care BedPro and FloorPro Alarms

There is no need to use the enclosed Velcro<sup>®</sup> strip in order to secure Skil-Care's BedPro or FloorPro alarm units to the bracket. Both these units have wire brackets that permit them to be mounted to the Bed Alarm Bracket and Belt using the following procedure:

1. Place the base (wide part) of universal bracket through the alarm unit's wire bracket. (Illustration C)
2. Tilt top of universal bracket up as you continue to place it through wire bracket. (Illustration D)
3. When universal bracket is fully inserted within the wire bracket, turn universal bracket around and place alarm unit on base. (Illustration E)
4. Reverse this procedure to remove alarm unit from Bed Alarm Bracket and belt.

