


## Tri-Fold Bedside Mat

Reorder #	Description	Size	Unit
911547	Tri-Fold Bedside Mat	1" x 26" x 68"	Each

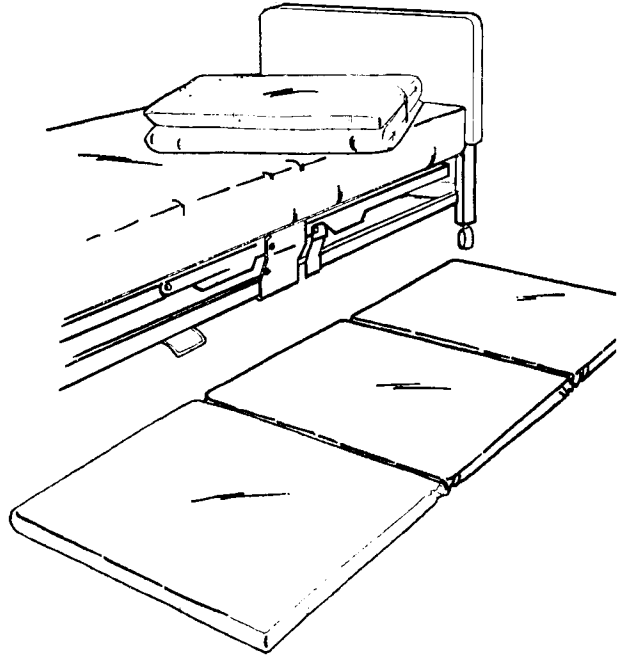

### PURPOSE

Placed on the left and right sides of the bed, Skil-Care's Tri-Fold Bedside Mat is intended to reduce the impact of a fall from bed. According to CMS, bed rails are restraints if used "to keep a resident from voluntarily getting out of bed as opposed to enhancing mobility while in bed." When bed rails cannot be used, CMS suggests "Placing the bed lower to the floor and surrounding the bed with a soft mat."

### INSTRUCTIONS FOR USE

Place the Tri-Fold Bedside Mat on the floor next to the bed. Two mats will be required if both rails are in the lowered position. **Important Notice: Make certain to place the Tri-Fold Bedside Mat on the floor with the label reading "THIS SIDE UP" facing up. The mat has a nonskid bottom that keeps it properly positioned next to the bed.**

### STORAGE

Skil-Care's Tri-Fold Bedside Mat is designed for easy storage. Simply fold the three sections and secure the Velcro<sup>R</sup> tabs. The mat fits easily in a closet or under the bed.

### SAFETY NOTICES

- It is the responsibility of each facility's appropriate medical professionals to determine if the Tri-Fold Bedside Mat is suitable as an injury-prevention device for a specific resident.
- To prevent staff members from stumbling, make certain all staff members, especially those working at night, are informed that the Tri-Fold Bedside Mat has been placed at the sides of the bed.

### CLEANING AND DISINFECTING

The Tri-Fold Bedside Mat may be cleaned using a spray-and-wipe cleanser. A mild disinfectant may be used.