

Safe-t mate

Product use and specification sheet

Model Number	TG-1
Brand Name	Tab Grabber®
Description	Tablet Computer / eReader Holder for Wheelchairs
Manufacturer	210 Innovations, LLC 210 Leonard Drive Groton, CT 06340 860-445-0210
Recommended Use	For wheelchair users desiring to have a tablet computer holder that attaches directly to their wheelchair frame.
Not recommended for	Wheelchair users who may tend to move unpredictably increasing the chance of injury by coming in contact with the unit. Wheelchair users who may tend to treat such attachments very roughly whereby breaking one or more components on the device.
Operation	To achieve greatest stability, the Tab Grabber mounting clamp is best attached to a vertical frame member on the wheelchair such as and armrest support. Once secured, the flexible gooseneck arm allows the unit to be swung to the side for ease of wheelchair entry and exit. At the same time, the arm is stiff enough to provide a firm position for touchscreen applications. The “head” of the Tab Grabber will swivel on multiple axis to provide optimal positioning. The tablet is mounted to the head by means of four retractable clips attached to elasticized cord that can be mounted to the corners or anywhere along the edge. Thin cases will work with the unit but thicker protective cases may prevent mounting clips from properly grabbing the tablet.
Specifications	Materials: Injection molded ABS plastic, Foam rubber, PVC plastic, steel, nylon covered shock cord. Fits: Round tubing from 5/8” diameter to 1 1/8” diameter. Weight capacity: 4 lbs. Tablet size capacity: Smallest- 3.5” X 5.5” Largest -8.5” X 11” Tablet thickness up to 1/2” at edges. Flexible arm length: 18” Minimum bend radius of flexible arm: 3.5”
Care	Wipe unit with cloth and mild spray cleaner such as glass cleaner. Do not submerge.